

TROSYD®

ONYCHODYSTROPHY Click Pen

NAIL CHANGES DUE TO TRAUMA OR PSORIASIS

1% HYDROLAC

Methylspermidine

TREATS NAIL CONDITIONS AND PREVENTS RELAPSES

ONYCHODYSTROPHY

Onychodystrophy refers to “nail changes”.

Therefore, it is a change in the normal shape of the nail plate, which becomes brittle, with layers that flake and have an altered appearance or colour. Onychodystrophy can be caused by endogenous (such as alterations of the nail matrix or of the structures surrounding the nail), and exogenous (i.e., external) factors. Among the most common of the latter are **lack of active ingredients**, such as minerals, including iron, which are useful for nail growth, or **traumatic episodes**, even minor, but repeated (microtraumas), which, for example, may result from sports activities or due to prolonged use of excessively compressive footwear. The nail appearance can therefore change not only in response to fungal or bacterial infections; for this reason, it is important to identify the cause of the change in appearance and apply the most suitable remedy for the condition at hand.

TROSYD® ONYCHODYSTROPHY CLICK PEN


Trosyd® Onychodystrophy Click Pen is a medical device for the treatment of nail changes caused by trauma, psoriasis, nutritional deficiencies and damage to the nail matrix, which cause yellowing, flaking and toe or hand nail detachment. Thanks to its effective formulation, Trosyd® Onychodystrophy Click Pen offers a **dual benefit**:

- **promotes the healing of the nail plate** because it deeply restructures the formation of keratin in the nail
- **protects from external aggressions**, because it creates a protective film on the nail plate, which, in turn, reduces the risk of infection

INSTRUCTIONS FOR USE

Before each application, clean the nail and remove any nail polish residues.

Follow these instructions:

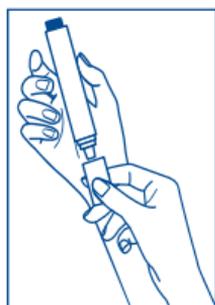

1) Remove the applicator's cap.



2) Hold the applicator and press the button at the top of the pen until the brush becomes moist.*



3) Distribute the product evenly on the entire nail surface and on the lower side of the nail edge, applying a gentle pressure.



4) Wait until the nail is dry (about 60-90 seconds) before you wear gloves, socks/stockings and/or shoes.

After use, carefully close the applicator to prevent the product from evaporating and the brush from hardening. Proceed with 1 application 2 times a week for at least three months. We recommend continuing the treatment cycle until the nail has completely healed.

* On first use it may take up to 20 clicks, while 1-2 clicks are enough when the pen is activated.

STORAGE INSTRUCTIONS

Store at room temperature, in a cool and dry place, away from light and heat sources.

WARNINGS

Keep out of children's reach. For topical nail use only. External use only. Do not swallow. In case of contact with eyes, rinse thoroughly. Do not use the device after the expiry date printed on the package. The expiry date refers to the product in intact packaging and correctly stored. Do not use if you are hypersensitive or allergic to one of the components. Carefully close the applicator after use. Discontinue treatment if you experience abnormal reactions. Do not use if package is opened or damaged.

INGREDIENTS:

Methylspermidine; Aqua, Hydrolysed Keratin; Panthenol; Glycolic acid; Acrylates/Ammonium Methacrylate Copolymer; Denatured alcohol
For information: info@giulianipharma.com

 Labomar S.p.A.
Via Nazario Sauro 35 I
31036 Istrana (TV) - Italy

Exclusive distributor: **Giuliani S.p.A.**
Via Pelagio Palagi, 2 - 20129 Milan (MI) – Italy



PAP
Information leaflet
Paper

*Sorted recycling.
Check the provisions in force
in your municipality.*

Rev. December 2020

2,5 ml    0477

