

TROSYD®

ONYCHODYSTROPHY

NAIL CHANGES DUE TO TRAUMA OR PSORIASIS

1% HYDROLAC

Methylspermidine

TREATS NAIL CONDITIONS AND PREVENTS RELAPSES

ONYCHODYSTROPHY

Onychodystrophy refers to “**nail changes**”.

Therefore, it is a change in the normal shape of the nail plate, which becomes brittle, with layers that flake and have an altered appearance or colour. Onychodystrophy can be caused by endogenous (such as alterations of the nail matrix or of the structures surrounding the nail), and exogenous (i.e., external) factors. Among the most common of the latter are **lack of active ingredients**, such as minerals, including iron, which are useful for nail growth, or **traumatic episodes**, even minor, but repeated (microtraumas), which, for example, may result from sports activities or due to prolonged use of excessively compressive footwear. The nail appearance can therefore change not only in response to fungal or bacterial infections; for this reason, it is important to identify the cause of the change in appearance and apply the most suitable remedy for the condition at hand.

TROSYD® ONYCHODYSTROPHY

Trosyd® Onychodystrophy is a medical device for the treatment of **nail changes due to trauma, psoriasis, nutritional deficiencies and damage to the nail matrix, which cause yellowing, flaking and nail detachment**. Thanks to its effective formulation, Trosyd® Onychodystrophy offers a **dual benefit**:

- **promotes the healing of the nail plate** because it deeply restructures the formation of keratin in the nail
- **protects from external aggressions**, because it creates a protective film on the nail plate, which, in turn, reduces the risk of infection

INSTRUCTIONS FOR USE

After cleansing the nails to be treated, proceed with 1 application twice a week for at least 3 months.

STORAGE INSTRUCTIONS

Store at room temperature, in a cool and dry place, away from light and heat sources.

WARNINGS

Keep out of children's reach. For topical nail use only. External use only. Do not swallow. In case of contact with eyes, rinse thoroughly. Do not use the device after the expiry date printed on the package. The expiry date refers to the product in intact packaging and correctly stored. Do not use if you are hypersensitive or allergic to one of the components. Close the bottle carefully after use. Discontinue treatment if you experience abnormal reactions. Do not use if package is opened or damaged.

INGREDIENTS:

Methylspermidine; Aqua, Hydrolysed Keratin; Panthenol; Glycolic acid; Acrylates/Ammonium Methacrylate Copolymer; Denatured alcohol

For information: info@giulianipharma.com

 Labomar S.p.A.
Via Nazario Sauro 35 I
31036 Istrana (TV) - Italy

Exclusive distributor: **Giuliani S.p.A.**
Via Pelagio Palagi, 2 - 20129 Milan (MI) – Italy

PAP 22
Information leaflet
Paper
Sorted recycling.
Check the provisions in force
in your municipality.

7 ml e   0477 Rev. December 2020

GIULIANI